SafeLine®

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SafeLine

Inhalt/Contents

Hygiene	Hygiene	4
Haut und Hände	Skin and Hands	5
Skin Des		6
Hand Des Protect		7
Lemon Wash		8
Instrumente	Instruments	9
Instru Des Direkt		10
Instru Des		11
Flächen-/Schnelldesinfektion	Surface-/Quick Disinfection	12
Surface Des		13
Surface Des Plus		14
Quick Des		15
Quick Des Lemon		16
Quick Des Wipes		17
Quick Des Wipes Lemon		18
Dentale Desinfektion/Reinigung	Dental Disinfection Cleansing	19
Suction Des		20
Suction Clean		21
Tray Clean		22
Prent Des		23
Zubehör Desinfektion	Disinfectant Accessories	24
Safeline Wipes		25
Zubehör/Desinfektionswannen	Accessories/Disinfection tubs	26
Spender	Dispenser	27
Informationen/Publikationen	Informations/Publications	28

Hygiene

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Hygiene

Hygiene is an issue which is subject to a multiplicity of laws, regulations, standards and recommendations from renowned institutions.

With the brand SafeLine, we have created a product which meets those criteria and in addition the price-performance ratio, is at least one step ahead of the competition. SafeLine will reliably ease and assist you in your daily work to place the greatest possible security into the working environment.

Hygiene measures, which when used properly, promote and protect the health of patients and health-care professionals for the long-term. This is where SafeLine places its importance and convinces through its quality.

We invite you, over the following pages, to get an image of SafeLine.

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Skin and Hands

Skin and Hand Disinfection

The hands are a major carrier of germs and pathogens. Hand hygiene is therefore the most important step for the prevention of infection disease. The washing and rinsing under running water namely reduces the germ-count, however does not kill off the pathogens.

Therefore, hygienic hand disinfection is required in all medical environments. The goal is to kill or deactivate mainly transmittable germs. The application times are to be considered here. The hands are to be kept wet for the duration of the application time.

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Special Advice:

All time specifications are minimum times. Depending on additional difficulties (e.g. moistly skin, dirty skin, high-risk surgical interventions) the exposure times should be elongated) SAFELINE Skin Des does not kill spores and cannot be used for the storage of sterile instruments and syringes.

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Skin and Hand disinfectant

2-propanol 70% (V/V) solution for the disinfection of hands and skin prior to injections, for cooling compresses

- bactericidal incl. TB bacteria
- fungicidal
- effective against all enveloped viruses as Hepatitis B and C virus, HIV, BVDV, Vaccinia and flu viruses as well as against Rotavirus
- spore-free filtered
- unscented

Application:

To be applied to the skin and for the preparation of cooling compresses.

Range of application:

Hygienic and surgical disinfection of hands. Skin disinfection prior to injections and peripheral vascular punctures. Skin disinfection prior to surgeries and punctures of joints. Disinfection of skin areas with high density of sebaceous glands. For cooling compresses.

Active ingredient in 100 g

Solution: 2-propanol 63,1 g

Application times:

Hygienic hand disinfection	30 sec.
Surgical hand disinfection	1,5 min.
Hand disinfection before injections, punctures	15 sec.
Hand disinfection before operations, punctures of joints, body cavities, hollow organs	
on sebaceous gland poor skin	1 min.
on sebaceous gland rich skin	10 min.
Bactericide incl. MRSA	
Fungicide	30 sec.
TB	1 min.
Efficacy against enveloped viruses incl. HBV/HIV/HCV/ Vaccinia/BVDV	30 sec.
Rota-Virus	15 sec.

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Alcoholic solution for the disinfection of hands

- with refatting ingredients
- perfume-free
- ready to use

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To be applied to the hands and forearms.

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Hygienic and surgical disinfection of hands, personal hand hygiene.

Active ingredient in 100 g

Solution: 80 g Ethanol, 0,1 g Benzalkoniumchlorid.

Application times:

Hygienic hand disinfection	30 sec.
Surgical hand disinfection	1,5 min.
Bactericide incl. MRSA, Fungicide acc. VAH/DGHM	30 sek.
TB	1 min.
Efficacy against enveloped viruses incl. HBV/HIV/HCV/ Vaccinia/BVDV	30 sek.
Rota-Virus	15 sek.
Novo-Virus acc. EN 14476	2 min.

Repeated washing of the hands affects the skin's protective barrier. Among other things, it can cause cracks and dehydration, which makes the penetration of germs and pathogens easier. Skin disinfection with moisturising agents is preventative in nature and supplies the skin with moisturising and skin-cleansing ingredients. High-quality conditioning oils are used for supporting and caring for the skin through its daily stresses.

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Tip:

Note proper implementation and problem areas such as fingers, thumbs and between fingers. Note exposure times. Keep your hands wet and if necessary, use multiple portions.

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nurturing washing lotion

Spares the body's own skin protective barrier and avoids the drying out of the skin.

- pH skin neutral
- contains nurturing substances
- pleasant lemon smell

Range of application:

Washing of skin and hands..

Ingredients:

Aqua, Sodium Laureth Sulfate, Sodium Chloride, PEG-7 Glyceryl Cocoate, Cocamide DEA, Sodium Cocosulfate, Phenoxyethanol, Parfum, 2-Bromo-2-Nitropropane-1,3-diol, Citric Acid, Limonene.

Note:

Preventive skin protection aims to avoid skin damage and contact allergies, which can arise when dealing with chemical substances, dusts and powders. The measures include protective gloves and separate, active skin care products.

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Hand care acts counter to the results of activity related stress and skin irritation caused by frequent washing and disinfecting as well as dealings with skin-damaging agents

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Instruments

Instrument disinfection/cleaning

Medical devices contaminated with germs, to be used invasively on people, can be a source of infection. Wide disinfecting effects, high cleansing power and a broad spectrum of action are basic requirements for instrument disinfectants and cleaners. The maximum reduction of hazardous materials allows for lower toxicity and promotes a better user experience. This results in a material-gentle and environmentally friendly product.

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Liquid solution for the disinfection and cleaning of instruments

- ready for use
- especially for rotating and dental instruments
- high degree of material compatibility

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Active ingredient in 100 g

Solution: 0,28 g Didecyl-methyl-poly(oxyethyl) ammonium propionate 0,19 g Guanidine, N,N''''-1,3-propanediylbis-, N-dodecyl, Diacetate

Exposure times:

bactericidal incl. MRSA, fungicidal (C. albicans) acc. to DGHM/VAH guidelines	15 min.
HBV, HIV, HCV, BVDV, Vaccinia (acc. to the guidelines of Robert Koch Institute)	5 min.

Tip:

Poor cleaning, insufficient water, inadequate disinfection and cleaning, incorrect concentrations and application times, a lack of care and handling errors are the most common causes of equipment damage and malfunctions.

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Concentrated disinfectant and cleaner for manual disinfection of instruments

Material-gentle aldehyde and phenol free instrument disinfectant with good cleaning power

- user-friendly active ingredients
- high degree of material compatibility and excellent cleaning power
- suitable for ultrasonic baths

Range of application:

Disinfection and cleaning of all kinds of medical and dental instruments. Also suitable for the reprocessing of thermoresistant and thermolabile instruments such as rigid endoscopes. Suitable for ultrasonic baths.

Active ingredient in 100 g

Solution: 5,63 g Didecylmethyloxyethyl ammonium propionate; 3,75 g Guanidine, N,N''''-1,3-propanediylbis-, N-coco alkyl derivatives, diacetates

Exposure times:

Bactericidal incl. MRSA, fungicidal (C. albicans) acc. to VAH/ DGHM-guidelines as well as TB bacteria (M. terrae)	
1%	60 min.
2%	30 min.
3%	15 min.

Effective against all enveloped viruses incl. HBV, HIV, HCV, BVDV, Vaccinia (acc. to the guidelines of Robert Koch Institute)

1%	30 min.
3%	15 min.

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5 Liter/litres Art.-No. 780033
Ref.-No. 09960

Tip:

A residue free instrument cleaning is the prerequisite for an effective disinfection and sterilization. Ultrasonic cleaning optimizes the cleaning results and is advantageous when the cleaning results cannot be assessed directly.

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Surface-/Quick Disinfection

To maintain general hygiene status, periodic surface disinfection is strictly necessary. Not following this form of disinfection creates a breeding ground for the development of pathogens.

The cleaning of surfaces, walls and floors is understood to be under general surface disinfection, which also applies to the range of machinery and equipment. The surface disinfectant has been created to cover these and other risk areas through highly effective disinfecting performance and cleaning power. Through short application times, a better material compatibility is developed and implemented.

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Concentrated disinfectant and cleaner for surfaces

Aldehyde and phenol free concentration for disinfection/cleaning of surfaces and medical equipment

- user-friendly active ingredients
- strong cleaning properties
- for all areas in the food industry
- suitable for acrylic glass
- perfume-free

Range of application:

Acc. to Medical Devices Act: disinfection and cleaning of medical devices .
Acc. to Biocide Directive: disinfection and cleaning of all kinds of surfaces and medical inventory. Also for all areas in the food industry and canteen kitchens.

Active ingredient in 100 g

Solution: 6,66 g Quaternary ammonium compounds, benzyl-C12-18-alkyldimethyl-, -chlorides; 3,33 g Dimethyldidecyl ammonium chloride

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5 Liter/litres Art.-No. 780022
Ref.-No. 09941

Tip:
Use solutions immediately. Use appropriate dosage aids and tables. Label containers.
Wash and disinfect with wiping cloths. Favour disposable cloths. Note preparation instructions and wear protective clothing

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Concentrated disinfectant and cleaner for surfaces

- Aldehyde and phenol free
- user-friendly active ingredients
- strong cleaning properties
- for all areas in the food industry
- suitable for PVC and linoleum floorings

Range of application:

Acc. to Medical Devices Act: disinfection and cleaning of medical devices.
Acc. to Biocide Directive: disinfection and cleaning of all kinds of surfaces and medical inventory.

Active ingredient in 100 g

Solution: 16 g Quaternary ammonium compounds, benzyl-C12-18-alkyldimethyl-, -chlorides

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5 Liter/litres Art.-No. 780024
Ref.-No. 09950

Tip:

For surface disinfectants, a dual status applies. Depending on the purpose, these are subject to the Biocides Act or the Medical Devices Act (MPG). Preparations which are used in the patient environment of medical areas are medical products and carry the CE mark. Surface disinfectants are subject to the Hazardous Substances Ordinance (GefStoffV). EU safety information sheets and operating instructions are necessary

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Alcoholic rapid disinfectant

- free from aldehydes
- ready for use
- for inaccessible surfaces

Range of application:

Acc. to Medical Devices Act: Rapid disinfection and cleaning of alcohol-resistant medical devices.
Acc. to Biocide Directive: Rapid disinfection and cleaning of alcohol-resistant surfaces and alcohol-resistant medical inventory.

Active ingredient in 100 g

Solution: 38,9 g Ethanol, 0,06 g Dimethyldidecyl ammonium chloride

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Bakterizid inkl. MRSA, fungizid (C. albicans) gem. DGHM/VAH-Richtlinien sowie TB-Bakterien (M. terrae)		*2 Min.
HBV, HIV, HCV, BVDV, Vakzinia (begrenzt viruzid gem. RKI-Empfehlung)		1 Min.

*gelistet in der 5-Minuten-Spalte

Exposure times:

bactericidal incl. MRSA, fungicidal (C. albicans) acc. to DGHM/VAH-guidelines as well as TB bacteria (M. terrae)		*2 min.
HBV, HIV, HCV, BVDV, Vaccinia (acc. to the guidelines of Robert Koch Institute)		1 min.

*listed in the 5-minute-column

Info:

Surface hygiene is an essential part of a hygiene plan and an important contribution to infection prevention. The basis for the determination of appropriate cleaning and disinfection measures are risk assessments. For officially ordered decontamination measures, the provisions under § 18, Infection Protection Act (IfSG) apply.

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10 Liter/litres Art.-No. 780002
Ref.-No. 09922

Tip:

Disinfectants are defined as effective killers/deactivators of pathogenic micro-organisms, so that surfaces and objects do not carry an infection risk.

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Alcoholic rapid disinfectant with Lemon flavour

- free from aldehydes
- ready for use
- for inaccessible surfaces

Range of application:

Acc. to Medical Devices Act:
Rapid disinfection and cleaning of alcohol-resistant medical devices.
Acc. to Biocide Directive:
Rapid disinfection and cleaning of alcohol-resistant surfaces and alcohol-resistant medical inventory.

Active ingredient in 100 g

Solution: 38,9 g Ethanol, 0,06 g Dimethyldidecyl ammonium chloride

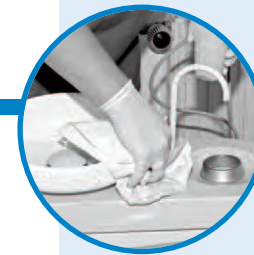
Info:

As cleaning is designated the removal of contaminants, a killing/deactivation of pathogenic micro-organisms takes place otherwise. A standard cleaning is sufficient for surfaces which cannot be contaminated and for surfaces without frequent hand or skin contact.

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Nachfüllpack/Refill
à 120 Tücher/Tissues
Art.-No. 780012
Ref.-No. 09931

Please notice:

Open the refill package. Form a roll from the opened refill package. Put the complete refill package into the box. Pull out the first tissue from the middle of the refill roll and pull it through the opening of the application system. Close the box duly. Please refill the box only with SafeLine Quick Des Wipes.

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*gelistet in der 5-Minuten-Spalte

Wet wipes for rapid disinfection with Lemon flavour

- free from aldehydes
- ready for use
- saturation liquid VAH-listed

Range of application:

Acc. to Medical Devices Act:
Rapid disinfection and cleaning of alcohol-resistant medical devices.
Acc. to Biocide Directive:
Rapid disinfection and cleaning of alcohol-resistant surfaces and alcohol-resistant medical inventory.

Active ingredient in 100 g Solution: 38,9 g Ethanol, 0,06 g Dimethyldidecyl ammonium chloride

Exposure times:

bactericidal incl. MRSA, fungicidal (C. albicans) acc. to DGHM/VAH-guidelines as well as TB bacteria (M. terrae)	*2 min.
HBV, HIV, HCV, BVDV, Vaccinia (acc. to the guidelines of Robert Koch Institute)	1 min.

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Dental Disinfection/Cleansing

Dentistry areas also pose health risks to be prevented. The commonly used disinfectant for the cleaning of surfaces, instruments and machines covers a large spectrum. Here however, there are special requirements placed on material characteristics and material compatibility for drills, extractors, spoons and moulds. The specially developed disinfectants and cleaners afford an optimal effect for users, patients, and materials.

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5 Liter/litres Art.-No. 780065
Ref.-No. 09980

Concentrated disinfection for dental suction systems

- material protecting and strong cleaning power
- low-foaming
- free from active chlorine

Application:
Use at least twice a day. Exposure time to be considered. After disinfection process rinse suction system with clear water. Dilute with warm water (40-50 °C) and extract by suction immediately. First fill in water, then the concentrate. Do not leave the disinfectant in the suction system over night.

Range of application:
Disinfection, cleaning and maintenance of dental suction systems. Regular use guarantees a proper hygienic condition of the suction system.

Active ingredient in 100 g Solution: 38,9 g Ethanol, 0,06 g Dimethyldidecyl ammonium chloride

Exposure times:	
Bactericidal incl. MRSA, fungicidal (C. albicans) acc. to DGHM/VAH guidelines 2%	15 min.
HBV, HIV, HCV, BVDV, Vaccinia (acc. to the guidelines of Robert Koch Institute) 2%	10 min.

Note:
For optimal material protection for the facility, it is absolutely recommended to perform the disinfection at the start of the work day and not to let the disinfectant stand overnight in the facility.

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Additional cleaner for dental suction systems

- strong cleaner power
- low foaming
- optimal material protection of the extraction unit

Application:
Use SafeLine Suction Clean one or two times a week. Depending on degree of pollution use 50 to 100 ml SafeLine Suction Clean for 1 liter working solution. For each cleaning process use 2 liters of freshly prepared working solution. Rinse suction system with clear water after exposure time has elapsed. Do not leave the working solution in the suction system over night!

Range of application:
For the regular, additional cleaning of dental suction systems, spittoon bowls and run-off tubes.

Exposure times:	
Depending on degree of soiling: 5 to 10% at 1.5 to 2 hours	
Composition:	
Organic acids, complexing agents, corrosion inhibitor, solubilizer	

Tip:
The best cleaning efficiency will be obtained with a contact time of approx. 1.5 to 2 hours. Upon termination of the contact time rinse with clear water in order to remove solved fouling completely in the tubes. Do not leave cleaning solution in the equipment over night.

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Anionische Tenside 5 - 15 %.

5 Liter/litres Art.-No. 780077
Ref.-No. 09985

Cleaning concentrate for dental trays

- removes alginate and dental cements in short time
- excellent material compatibility
- strong cleaning power

Application:

Depending on degree of soiling prepare a 5-10 % working solution (50 to 100 ml concentrate to be filled up with water to 1 liter). Warm water (max. 40° C) enhances the cleaning properties. Pre-clean trays and instruments first and then immerse them in the working solution. Let the solution act for some hours or over night. Remove dirt particles from trays and instruments under flowing water by using a suitable brush or scrubber.

Range of application:

Self-acting cleaner for dental trays. Strong cleaning power to remove alginate, adherence material and dental cements. Suitable to clean dental instruments made of plastics, stainless steel and anodized aluminium. Can be used in ultrasonic bath.

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5 Liter/litres Art.-No. 780076
Ref.-No. 09983

Liquid solution for disinfection and cleaning of dental impressions and prosthetic dentistry materials

- ready for use
- does not influence dimensional accuracy
- no corrosion of surfaces

Application:

Immerse dental components made of alginates and hydrocolloids shortly and then let them dry for 10 minutes. Other materials please immerse for 10 minutes and rinse them with clear water afterwards.

Range of application:

Ready for use solution for a rapid disinfection and efficient cleaning of dental components and prosthetic dentistry material such as dental impressions made of polyethers, A/C silicones, alginates and hydro-colloids.

Active ingredient in 100 g

Solution: benzalkoniumchloride 0,48 g; Ethandial 0,20 g; Glutaraldehyde 0,01 g

Exposure times:

Bactericidal incl. MRS, fungicidal (C. albicans) acc. to DGHM/VAH-guidelines tested at high bio burden level	5 min.
HBV, HIV, HCV, BVDV (acc. to the guidelines of Robert Koch Institute)	10 min.
Vaccinia virus	10 min.

Note:

According to UVV, wear protective gloves when handling particular disinfectants which should not come in contact with skin. Avoid skin and eye contact with the solution. Wash contaminated skin with plenty of clean water immediately. Keep out of reach of children. Do not ingest.

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Disinfectant Accessories

In order to work optimally, you need the right tools.

Arrive at the correct dosage, so that consumption and disinfection success are in balance.

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Dispenser Systems for Sterilising Solution

Our Petsorb™ reels, PET fibre blend, were designed for an ideal active substance release in combination with the Safeline® sterilisation solution. The robust, wet strength and lint free wet cloths made from high-quality fleece, possess an excellent adsorption performance so that a sufficient dispensing of the sterilisation substance will occur.

The wet cloths are hygienically packaged separately in a transparent bag with label.

The wet cloths are available with or without hole-structure.

The dispensing buckets are also available in our product range.

1 - D-Wipes Small Petsorb™
for 3,4 Litres bucket
Quantity: 70 sheets
Grammage: $\pm 50 \text{ g/m}^2$
Sheet size: approx. 25 x 25 cm
Structure: Finely perforated

2 - V-Wipes Petsorb™
for 6,2 Litres bucket
Quantity: 90 sheets
Grammage: $\pm 55 \text{ g/m}^2$
Sheet size: approx. 37 x 20 cm
Structure: Smooth non-perforated

3 - M-Tex-Wipes Petsorb™
for 6,2 Litres bucket
Quantity: 60 sheets
Grammage: $\pm 50 \text{ g/m}^2$
Sheet size: approx. 30 x 34 cm
Structure: Rough perforated

4 - Refill bucket 3,4 Litres
made of high quality plastic
Content: 3,4 Litres
Height: 156 mm

5 - Refill bucket 6,2 Litres
made of high quality plastic
Content: 6,2 Litres
Height: 212 mm

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Canister dosage pump
with 30 ml dosage per lift pres-
sure, fits for 5 or 10 Litres canister

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Canister Key
for easy opening. Fits on to 5; 10
and 30 Litres canister

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Disinfection tubs
suitable for for all Disinfection products
Set with tub, lid and strainer
High quality plastic, white

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Canister dosage pump
fits for 500/1000 ml
bottles

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Decanting tap
for 5, 10 and 30 Litres canister

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Wall mounted dispenser metal
with housing made of aluminum, pump
and oscillating arm made of stainless
steel. Autoclavable.
Delivered ready to install.

Flaschengröße Size of bottle	Armhebel Oscillating arm	Art.-No.	Ref.-No.
500 ml	kurz/short	780086	09993-KH
500 ml	lang/long	780087	09993-LH
1000 ml	kurz/short	780088	09994-KH
1000 ml	lang/long	780089	09994-LH

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Wall mounted dispenser metal
with housing made of aluminum,
oscillating arm made of stainless steel.
Autoclavable.
Delivered ready to install.

Flaschengröße Size of bottle	Armhebel Oscillating arm	Art.-No.	Ref.-No.
500 ml	kurz/short	780082	09991-KH
500 ml	lang/long	780083	09991-LH
1000 ml	kurz/short	780084	09992-KH
1000 ml	lang/long	780085	09992-LH

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Wall mounted dispenser plastic
Oscillating arm made of high quality
plastic.
Delivered ready to install.

Flaschengröße Size of bottle	Armhebel Oscillating arm	Art.-No.	Ref.-No.
500 ml	lang/long	780080	09990-500
1000 ml	lang/long	780081	09990-1000

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Disinfection plans for medical and dental practices as well as for nursing and rest homes according to the legal specifications, as well as data sheets, safety data sheets, operating instructions and certificates can be requested from your responsible salesman.

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Infection Protection Act:

§ 18 IfSG: officially ordered disinfection and disinfestations

§ 36 IfSG: compliance with infection hygiene

Recommendations of Robert-Koch-Institute:

- Requirements for hygiene on preparation of medical products
- Requirements for hygiene on preparation of flexible endoscopes and endoscopic additional equipment
- Requirements for hygiene on the structurally functional design and instrumentation of endoscope units
- List of certified and approved disinfectants and disinfecting processes
- Examination and declaration of effectiveness of disinfectants against viruses

Rules and Regulations of the Professional Association:

BGV A 1:	Accident provision policy of prevention
BGR 250/TRBA 250:	Biological substances in public health sector and in public welfare
BGR 198:	Use of protective clothing
BGR 195:	Use of protective gloves
BGR 206:	Disinfection operations in medical service

28

Safeline